


Remote Plus app

You choose how
you want to hear
your world

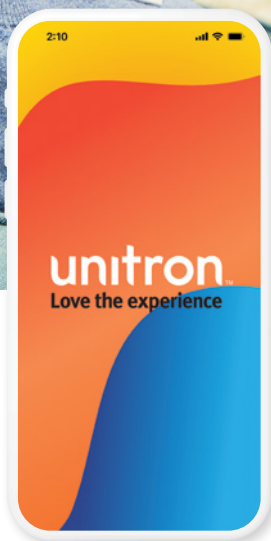

A Sonova brand

Embrace your hearing journey for what it truly is: **it's your experience**

And when you do, you'll discover that the Remote Plus app makes it easy for you to customize your hearing journey your way, every step of the way.

*For the use of the Remote Plus app, Unitron hearing aids with Bluetooth® connectivity are required. The Remote Plus app works on Apple smartphones with iOS 15 or newer and Android smartphones with version 8.0 or newer.



With quick and seamless navigation in the Remote Plus app, you can easily and discretely make the adjustments you need in the moment. From volume control to programs that you can select and customize, you choose how to personalize your experience!

You can even customize the app's user interface by choosing between two modes to suit your desired level of functionality.

Get started in just a few taps


1



First, search for Unitron Remote Plus on the App Store or Google Play or scan the QR codes below, and download the app to your smartphone.




2

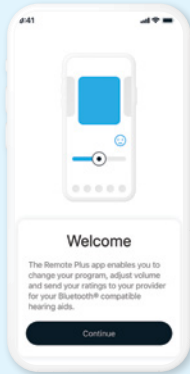

Launch the app and follow the on-screen prompts to pair your hearing aids, and you are ready to go!


App Store badge is a trademark of Apple Inc., registered in the U.S. and other countries. Google Play and the Google Play logo are trademarks of Google LLC.

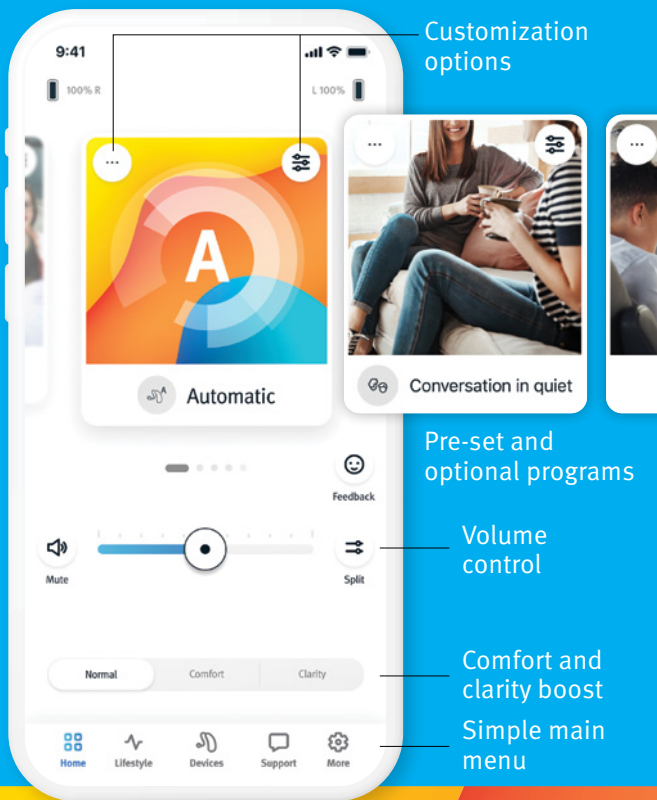

Personalize your experience in any moment


The Remote Plus app allows you to adjust sound **clarity** and listening **comfort** in the moment. From the home screen, you can swipe to select pre-set and optional programs designed for different locations, times of day, conversations or lifestyle activities.

Tap **customization options** to personalize the look and feel of your program carousel, and to access sound and equalizer controls for further optimization of your listening experience.

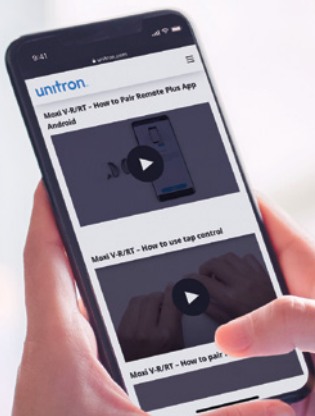

Take confident steps into better hearing

It might feel like there is a lot to learn at your first appointment, but don't worry about remembering it all. As you begin your hearing journey, you can confidently manage the day-to-day maintenance of your hearing aids with the help of the app-based **Coach** notifications. You'll have access to easy-to-understand instructions and videos, plus you'll receive reminders and tips delivered straight to your smartphone.

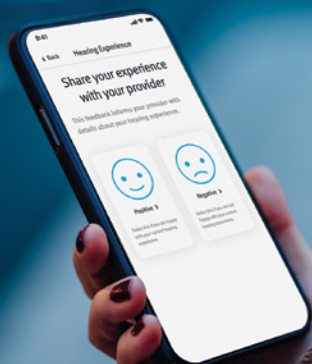

Share your experience in real time

Stay in touch with your hearing care professional with **Ratings**. You can inform them of your successes and challenges as they occur so you don't have to wait until your next appointment. Your hearing care provider can even send some adjustments directly to your smartphone for you to accept and apply improvements to your listening experience in real time.

unitron™ **Love the
experience**

At Unitron, we empower people with life-enhancing hearing experiences that fit seamlessly into their world. Our sound performance technology, Experience Innovations, and intuitive design work perfectly together to provide personalization and optimization. Because everyone deserves to **Love the experience.**™

© 2023 Sonova AG, or its affiliates.

028-6502-02/V2.00/2023-12

unitron.com

CE
0459



Sonova AG,
Lauderhofstrasse 68
2610, CH-8212
Olten, Switzerland



sonova
HEAR THE WORLD